

**DIRECTIONS – ISTRUZIONI – PRÉPARATION - GEBRAUCHSANWEISUNG-  
INSTRUCCIONES – INSTRUKTIONER – INSTRUÇÕES - ΟΔΗΓΙΕΣ****ChromArt ESBL SUPPLEMENT****REF** 4240080

10 X 5 mL

**IVD****CE****INTENDED USE**

Freeze-dried selective supplement to be used with CRE-ESBL Base (REF 4080252) for the detection of Extended Spectrum Beta Lactamase (ESBL) producing *Enterobacteriaceae* in clinical specimens.

**VIAL CONTENTS**

Antimicrobials mix.....0.21 g/vial

**CONTENTS/CONTENUTO/PRESENTATION/PACKUNGSINHALT/CONTENIDO/ INNEHÅLL /CONTEÚDO/ ΠΕΡΙΕΧΟΜΕΝΑ**

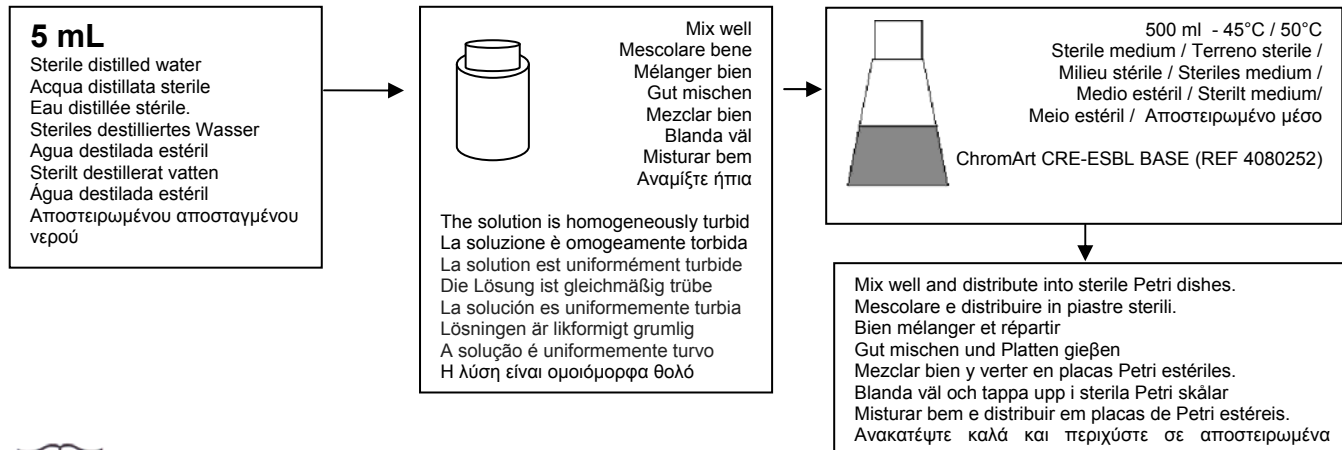
10 VIALS FOR 500 ML OF MEDIUM / 10 FLACONI PER 500 ML DI TERRENO / 10 FLACONS POUR 500 ML DE MILIEU / 10 RÖHRCHEN FÜR 500 ML NÄHRBODEN / 10 VIALES PARA 500 ML DE MEDIO / 10 AMPULLER, VARJE AMPULL ANVÄNDS TILL 500 ML MEDIUM/ 10 FRASCOS PARA CADA 500 ML DE MEIO / 10 ΦΙΑΛΙΔΙΑ, ΕΚΑΣΤΟ ΓΙΑ 500 ML ΜΕΣΟΥ.

**STORAGE / CONSERVAZIONE / CONSERVATION / LAGERUNG / CONSERVACIÓN / LAGRING / CONSERVAÇÃO / ΑΠΟΘΗΚΕΥΣΗ**

Store at 2-8° - When stored as directed the supplement remains stable until the expiry date shown on the label. Do not use beyond stated expiry date. / Conservare a 2 - 8°C- In queste condizioni il prodotto rimane valido fino alla data di scadenza indicata in etichetta. Non utilizzare dopo la data di scadenza. / Conserver à 2-8°C, selon les conditions de conservation indiquées, le produit est stable jusqu'à la date d'expiration mentionnée sur l'étiquette. Ne pas utiliser après la date d'expiration. / Lagerung: 2-8°C. Das Supplement st bei vorschriftsmäßiger Lagerung bis zum aufgedruckten Verfalldatum haltbar. Verfalldatum beachten. / Conservar entre 2-8°C. Estable bajo estas condiciones, hasta la fecha de caducidad que figura en la etiqueta. No utilizar si ha caducado. / Lagras i 2-8°C. Om lagring sker korrekt håller sig supplement hela tiden fram till utgångs datumet som står antecknat på etiketten. Använd ej produkten efter utgångs datum. / Armazenar a 2-8°C. Quando armazenado nestas condições, o suplemento permanece estável até a data de expiração indicada no rótulo. Não utilizar após a data do vencimento. / Φυλάξτε στους 2-8°C. Όταν φυλάσσεται σύμφωνα με τις οδηγίες, το συμπλήρωμα παραμένει σταθερό μέχρι την ημερομηνία λήξης που φαίνεται στην ετικέτα. Μην χρησιμοποιείτε πέραν της αναφερομένης ημερομηνίας λήξης.

**PRECAUTIONS / PRECAUZIONI / PRÉCAUTIONS / VORSICHTSMABNAHMEN / PRECAUCIONES / SÄKERHETSFÖRESKRIFTER  
PRECAUÇÕES / ΠΡΟΦΥΛΑΞΕΙΣ**

For *in vitro* diagnostic use only. The supplement should be used only by adequately trained personnel with knowledge of microbiological techniques in the laboratory. / Solo per uso diagnostico *in vitro*. Il prodotto deve essere usato solo in laboratorio, da operatori addestrati e con conoscenze delle tecniche microbiologiche di base. / Pour usage *in vitro* exclusif. Ce produit ne doit être utilisé qu'en laboratoire par des personnels formés aux techniques de microbiologie. / Zum Gebrauch für *in vitro* Diagnostik. Nur entsprechend ausgebildete Personen mit Kenntnissen in mikrobiologischen Methoden dürfen dieses Produkt nutzen. / Uso sólo para diagnóstico *in vitro*. El suplemento debería ser sólo usado por personal adecuadamente cualificado con conocimientos de las técnicas microbiológicas en laboratorios. / Endast för *in vitro* diagnostik. Supplementet skall endast hanteras av behörig personal med kunskap om mikrobiologiska laboratorietekniker. / Somente para uso diagnostico *in vitro*. O suplemento deve ser utilizado somente no laboratório por pessoas adequadamente treinadas e com conhecimento em técnicas microbiológicas. / Το συμπλήρωμα πρέπει να χρησιμοποιείται από κατάλληλα εκπαιδευμένο προσωπικό με γνώση των μικροβιολογικών τεχνικών στο εργαστήριο.

**DIRECTIONS / ISTRUZIONI / PREPARATION / ZUBEREITUNG / INSTRUCCIONES / INSTRUKTIONER / INSTRUÇÕES / ΟΔΗΓΙΕΣ.**

www.biolifeitaliana.it