

DERMATHOPHYE ANTIMICROBIC SUPPLEMENT

Vial contents (for 500 ml of medium)

Chlortetracycline HCl 50 mg

DIRECTIONS

Aseptically reconstitute the contents of one vial with 5 ml (or 50 ml) of sterile distilled water. Prepare 500 ml (or 5 l) of Dermatophyte Selective Medium (cat. N° 4013691), autoclaved and cooled to 50°C. Add the supplement, mix well and distribute into sterile tubes or plates.

INTENDED USE

The supplemented medium corresponds to Taplin's, Zaias and Rebbel's formula and is used for the rapid, selective isolation and differentiation of the dermatophytes fungi responsible for lesions of the skin, nails and hair. For the details of the procedures see the technical sheets of Dermatophyte Selective Medium, cat. N° 4018691.

STORAGE

Store at 2-8° - When stored as directed the supplement remains stable until the expiry date shown on the label. Do not use beyond stated expiry date.

PRECAUTIONS AND WARNING

For *in vitro* diagnostic use only. The supplement should be used only by adequately trained personnel with knowledge of microbiological techniques in the laboratory.

Consult the material safety data sheet before the use.

PACKAGING

4240024 Dermatophyte Antimicrobial Supplement 10 vials, each for 500 ml of medium