

UREA, 40% SOLUTION

VIAL CONTENTS

Urea.....2 g
Distilled water.....5 ml

DIRECTIONS

The Urea Solution is ready to use. Prepare Urea Agar Base or Urea Broth Base according to the directions described on the label. Add 5 ml of Urea Solution to 100 ml of Urea Base Media cooled to 50°C the agar, and at room temperature the broth. Both the complete media are used for the determination of the urease activity of the organisms. For the details of the procedure see the technical sheet of Urea Agar Base and Urea Broth Base (Cat. N° 402175, 402180).

STORAGE

Store at 2-8° - When stored as directed the supplement remains stable until the expiry date shown on the label. Do not use beyond stated expiry date.

PRECAUTIONS AND WARNING

For *in vitro* diagnostic use only. The supplement should be used only by adequately trained personnel with knowledge of microbiological techniques in the laboratory.

PACKAGING

4240096 **Urea, 40% Solution, 10 x 5 ml**